

Protocol

Electrocardiography (ECG)

Method

Equipment:

Set of standard limb electrodes, 6 chest single-use electrodes with connectors to cables or electrodes with fastening belt, connecting cables, PC, ECG gel, cotton-wool, examination bed, ECG ruler.

Procedure:

Clean the skin of the examined subject with alcohol and spread ECG contact gel on the electrodes. Place the single-use electrodes at the corresponding locations. Connect all electrodes by connecting cables, which are marked with the respective letter and – in the case of limb electrodes – the respective color.

The places for fixing the electrodes

Limb leads:	red electrode:	right hand wrist
	yellow electrode:	left hand wrist
	green electrode:	left foot
	black electrode:	right foot
Chest leads:	V ₁	4 th intercostal space right from the sternum
	V ₂	4 th intercostal space left from the sternum
	V ₄	5 th intercostal space in the left medioclavicular line
	V ₃	between V ₂ a V ₄
	V ₅	5 th intercostal space in the left front axillar line
	V ₆	5 th intercostal space in the left medial axillar line

Working with PC:

1. Launch the application ECG-Seiva (the icon with a heart)
2. Press **Ins** and set the name and surname of the experimental subject; the other values are not relevant for this practical. Confirm by pressing **CTRL+Enter**. Press twice “**Beru na vědomí**”.
3. By pressing **F₄** activate ECG recording.
4. Press **F₄** to start ten-second ECG recording; it is necessary for the experimental subject to stay calm. Recording will terminate automatically.
5. Print the record by pressing **F₆**.
6. To return to the main menu, press **Alt+F₄**. To examine the next experimental subject, repeat the procedure from step 2.
7. Terminate the program by repeatedly pressing **Alt+F₄**

Name:

Numer:

Study group:

Evaluating an ECG record:

1. Heart rhythm:

- **Regularity:** Is the heart rhythm regular?
 - YES (heart rhythm is regular)
 - NO (heart rhythm is irregular)
- **Rhythm source:** Is each QRS complex preceded by P wave?
 - YES (sinus rhythm)
 - NO (idiopathic rhythm)

2. Heart rate:

- from PC:.....
- calculation from paper speed (25 mm/s).....
- evaluation according to ECG ruler.....

3. Duration of:

- RR interval.....
- PQ interval.....
- QRS complex.....
- QT interval.....

Challenge: estimate the Sokolow-Lyon index [S wave (V_1 or V_2) + R (V_5 or V_6)]:


.....
(> 35 mm = left ventricle hypertrophy)

Name:

Numer:

Study group:

Construct **electrical axis** of the heart from two limb leads into the Einthoven triangle and measure its angle in frontal plane. Assess its angle in horizontal plane from chest leads.

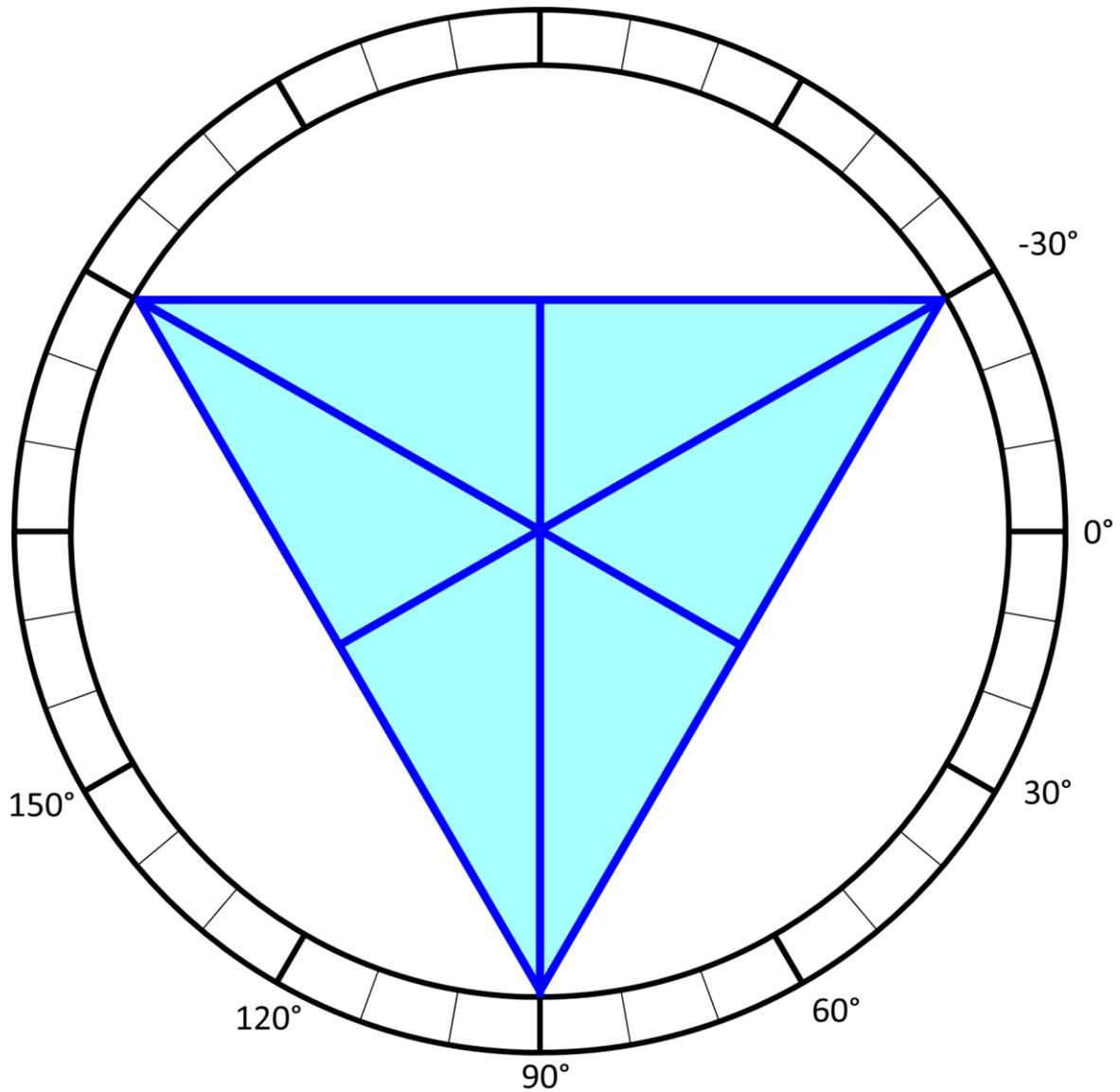

Conclusion

.....

.....

.....

.....

.....

.....

.....

.....

.....